


# Quick Start Instructions

## 1. Insert 9V Battery


System comes with factory settings: Full Volume and Tone 1.

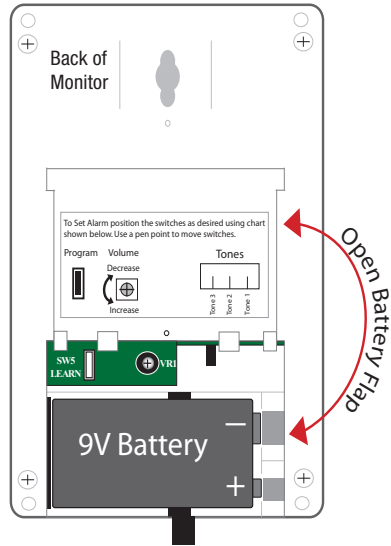

Note: Black ribbon goes underneath battery for easy removal.

## 2. Push "Learn" Button

Alarm will beep 3 times

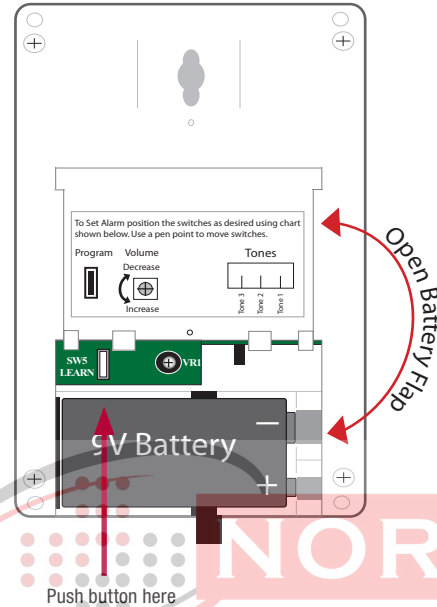


## 3. Plug in Transmitter

You now have 30 seconds to plug in transmitter to cord coming out of pad. Switch Transmitter On/Off button to "On". Alarm will make one long beep and is now programmed. Close neck of pad with button snaps. (Also see instructions on pad).



## 4. Test System

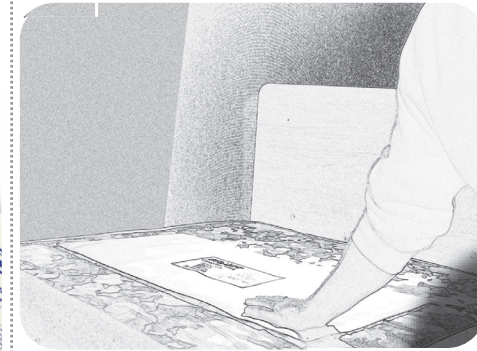


Apply pressure to pad to activate alarm. 2 beeps indicates alarm is functioning.

Alarm will sound when pressure is released from pad. Press "reset" button on front of monitor to silence alarm.


"Status/In Use" light blinks only when pad is plugged into monitor and pressure is on pad.

## 5. Mounting Options

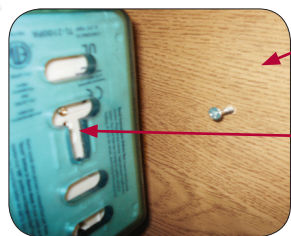

Drill hole into headboard or wall.

Hang monitor on screw using mounting hole on back of monitor.

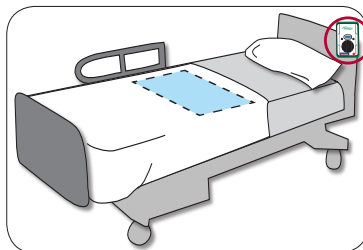


Pad and monitor can be up to 40 ft. away from each other

Note: Other mounting options available. Call 800.650.3637 for more information.


## Adjust Settings (Optional)

System comes with factory settings: full volume, tone 1. Use a small flathead screwdriver to adjust volume and a pen point to adjust tones.

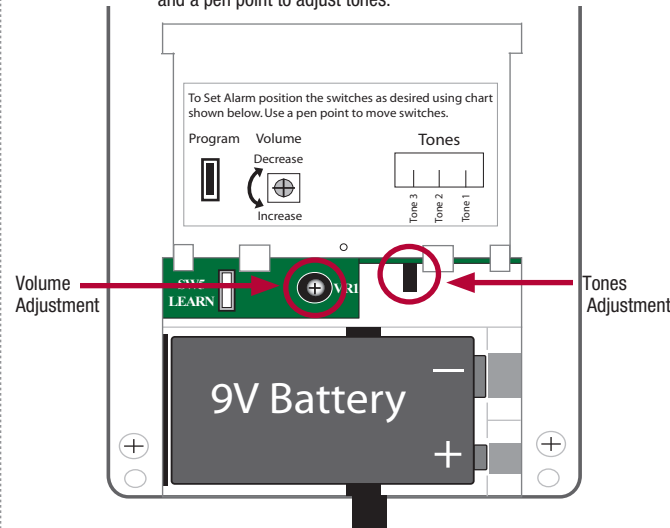

## Tamper Proof/Caregiver Key Mode

Use Caregiver Key to allow only the Caregivers to silence the alarm! This will cancel the function of the "Reset" button.

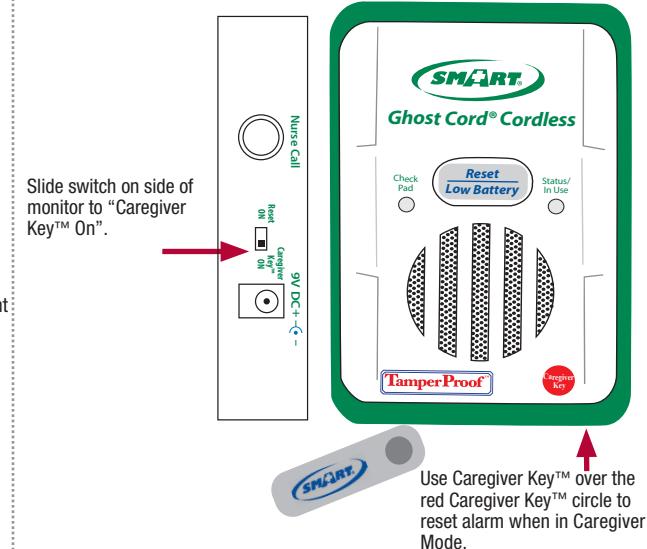



## TL-2100G Ghostcord® Cordless Monitor

### Installation Instructions

## TL-2100G Ghostcord Cordless® System


#### BED PLACEMENT

- Place pad under sheet.
- Place pad under shoulders to be alerted when resident sits up in bed.
- Place pad under lower back/buttocks to be alerted when resident vacates the bed.


#### TEST SYSTEM

- Apply pressure to pad to activate alarm. Monitor will beep twice when active.
- Alarm will sound when pressure is released from pad. Press "reset" button on front of monitor to silence the alarm. The status light on the front of monitor will blink when pressure is applied to pad.

#### Bed Pressure Pads

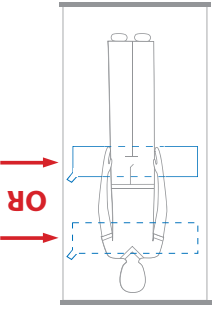

OR

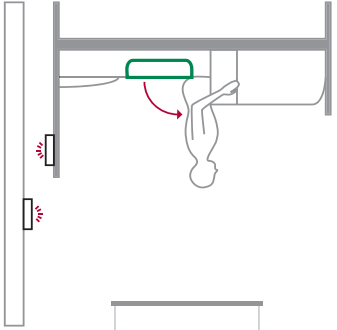


#### WHEELCHAIR/CHAIR PLACEMENT

- Center pad under buttocks.
- Place monitor out of reach of resident. Suitable mounting locations include: back of headboard, back of wheelchair, wall or under bed.


#### TEST SYSTEM

- Apply pressure to pad to activate alarm. Monitor will beep twice when active.
- Alarm will sound when pressure is released from pad. Press "reset" button on front of monitor to silence the alarm. The status light on the front of monitor will blink when pressure is applied to pad.

#### Wheelchair Pressure Pads

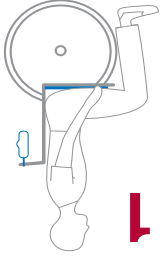

1

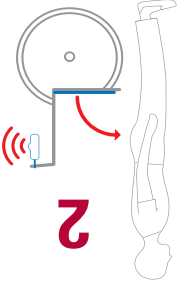

2

Need Set-up Help? (800) 650-3637



Warning: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is to help augment caregiver comprehensive resident mobility management program.

The Smart Caregiver Corporation GHOSTCORD® monitors are designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

a.) Direct patient supervision  
 b.) Adequate training for staff personnel for elopement  
 c.) Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

**WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.**

Smart Caregiver Corporation  
 sales@smartcaregivercorp.com  
 Toll Free: 800-650-3637

