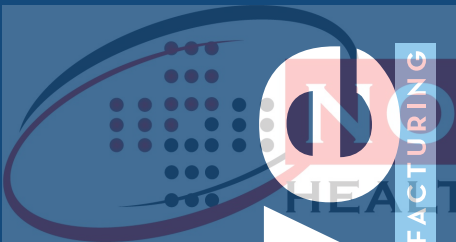
[www.drivemedical.com](http://www.drivemedical.com)

drive

MEDICAL DESIGN & MANUFACTURING

duet transport chair / rollator

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EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

[www.drivemedical.com](http://www.drivemedical.com)

drive

MEDICAL DESIGN & MANUFACTURING

## assembly instructions

Do not attempt to adjust or operate the Duet without carefully reading all instructions in this manual.

1. Remove all contents from carton: backrest, handle with tube, Rollator frame, wheels and pouch.
2. Open frame and slide padded backrest into frame brackets making sure snap buttons protrude through holes. For use as a rollator, reference Figure 1. For use as a transport chair, reference Figure 2.
3. Unscrew tightening knob from handle and insert handle tube into frame at desired height lining up holes appropriately. Slide bolts through holes from the outside and tighten knob until secure.
4. Lay rollator on floor so seat is facing the floor and unfold the legs. Straighten the legs until the side hinges lock into place.
5. Attach pouch by securing snap buttons around frame.

Rollator Position

figure 1

Transport Chair Position

figure 2

**WEIGHT CAPACITY: 300 LBS**

**THIS ITEM SHOULD NEVER BE SECURED FOR TRANSPORT.**

## safety precautions

1. Please make sure the Duet is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Duet for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Duet. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.
6. Before entering or leaving the chair, engage the dual wheel locks against the tire on both rear wheels. Make sure the front casters are in the forward position before transferring into or out of the chair.
7. Leaning forward while using the Duet as a transport chair with out proper assistance can cause tipping.

**NOTE:** You and your attendant should consult a qualified professional for the correct techniques on ramp negotiations.

## loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. When standing in a stationary position the user must pull up on the loop lock handle to hold the rear wheels in place.
2. To Temporarily Lock Wheels in Position – Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks – Pull up on both loop locks with palms of hand to original position.

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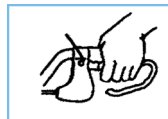
## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**drive**  
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.