### SAVE THESE INSTRUCTIONS

Yes, you can.

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

#### ⚠ WARNING

- **DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- All four leg tips **MUST** be in contact with shower/tub floor at all times.
- **ALWAYS** inspect the shower chair to ensure that it is properly positioned and stable before using. **DO NOT** use if shower chair is wobbly or unstable.
- Ensure that the snap buttons fully protrude through their respective adjustment holes.
- Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips **IMMEDIATELY**.
- Users with limited physical capabilities should be supervised or assisted when using the shower chair.
- The shower chair is not to be used as a transfer bench, transfer device or climbing device.
- Exercise caution when assembling the chair to avoid pinching.
- After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware and parts are secure.
- Ensure that all four chair legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

#### ⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

## Shower Chair Models

MODEL	DESCRIPTION	WEIGHT LIMITATION
91-2	Shower Chair	315 lb (143 kg)
95-2	Shower Chair with Back	300 lb (136 kg)
96-2	Shower Chair with Back	315 lb (143 kg)
9981	Folding Shower Chair with Back	250 lb (114 kg)
9785-1	Bariatric Shower Chair with Back	700 lb (318 kg)
9785-2	Bariatric Shower Chair with Back	700 lb (318 kg)

## Assembling the Shower Chair

NOTE: Not all procedures apply to all chair types. Refer to the chair type to verify if the procedure applies before attempting any procedure.

NOTE: For this procedure, refer to FIGURE 1.

- For models 96-2 (DETAIL "A") or 9785-1 and 9785-2 (DETAIL "B"):
  1. Put the shower chair on its side on a flat, stable surface.
  2. Press the snap buttons on the back support tubes and slide the back support tubes into the back support brackets on the leg frame until both snap buttons are fully engaged inside the holes in the back support brackets.

NOTE: Two audible "clicks" will be heard.

- For model 95-2 (DETAIL "C"):
  1. Place the seat upside down on a flat, stable surface.
  2. Put a leg over one socket in the seat and align.
  3. Insert the leg into the socket until it is fully seated and the socket tabs snap into the socket tab windows.
  4. Pull up on the leg to ensure that it has "locked" into place.
  5. Repeat STEPS 1-4 for the remaining three legs.
  6. Put the back into the slot in the seat.
  7. Push down on the back until it snaps securely in place.
  8. Ensure the compression buttons are fully visible through the notches in the back of the seat.

NOTE: If both compression tabs are visible, the back is installed properly. If the compression tabs are not visible, apply pressure to the back until it snaps into place and both compression tabs are visible.

## Adjusting the Shower Chair Height

NOTE: For this procedure, refer to FIGURE 2.


1. Press the snap button and slide leg extension up or down to desired adjustment hole.
2. Ensure the snap button protrudes fully through the adjustment holes of the leg extension.

NOTE: There will be an audible click when the snap button fully engages.


3. Repeat STEPS 1 and 2 for the other legs.

NOTE: Make sure that all four legs are adjusted to the same height.

DETAIL "A" - MODEL 96-2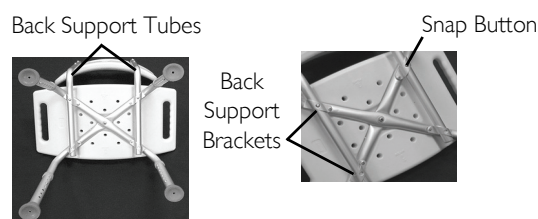

DETAIL "B" - MODEL 9785-1 AND 9785-2

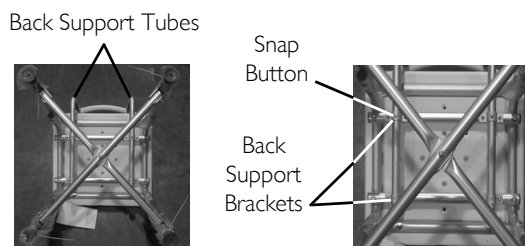DETAIL "C" - MODEL 95-2

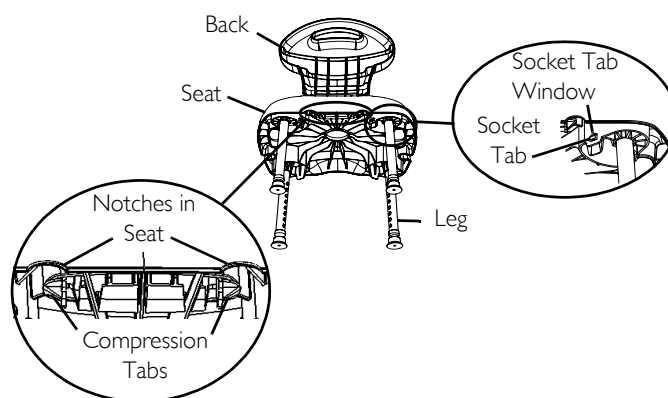
FIGURE 1 Assembling the Shower Chair

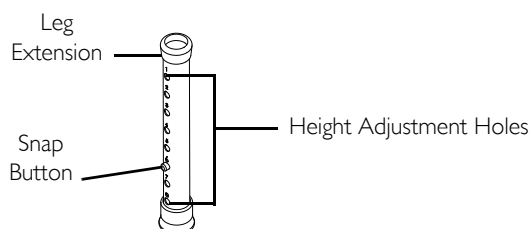
FIGURE 2 Adjusting the Shower Chair Height

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

# Unfolding and Folding the Shower Chair

NOTE: This procedure applies to shower chair model 9981 only.

NOTE: For this procedure, refer to FIGURE 3.

1. Place the shower chair on the floor/tub floor in the upright position.
2. Pull on the top edge of the inside leg to unlock the pivot stop bracket from the recess in the underside of the seat (Detail "A").
3. Lift up on rear edge of seat.

NOTE: When folding the shower chair, the pivot stop bracket MUST be flipped up BEFORE the pivot stop bracket can lock into the recess (Detail "B").

4. Grab the top of the inside leg and pull towards the rear of the seat until stopped by the pivot stop bracket (Detail "C").
5. Lower the rear edge of the seat so that the top of the inside leg fits securely into the groove along the back edge of the seat (Detail "D").

## ⚠ WARNING

Ensure the top of the leg tube is fully inserted and seated in the seat groove before using this device.

6. If necessary, push the backrest into position (Detail "E").

## Cleaning

NOTE: Before using any cleaning products, test an area on the bottom of the shower chair to make sure the product does not cause discoloration or staining. DO NOT use an abrasive detergent or cleaner to clean the shower chair.

Use a water and a mild, non-abrasive cleaner to clean the shower chair.

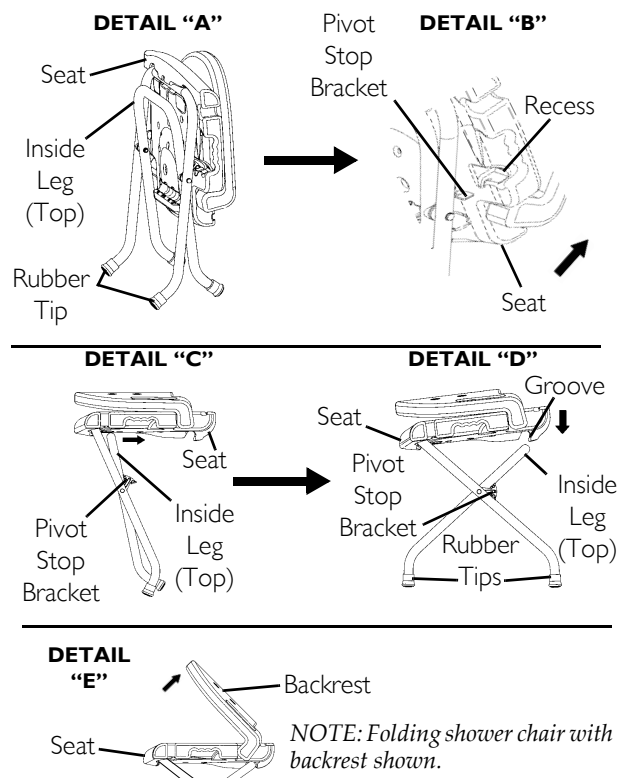
FIGURE 3 Unfolding and Folding the Shower Chair

## Limited Warranty

Invacare warrants this product to be free from defects in materials and workmanship.

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
© 2009 Invacare Corporation