DEALER: These instructions MUST be given to the user of this product.

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

#### ⚠ WARNING

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Grab Bars add security for bath and commode areas when attached as recommended. To provide optimum safety and function, location, size, and angle of bar(s) **MUST** be determined by a qualified professional.
- Use **EXTREME** caution on wet tub or floor surfaces.
- Users with limited physical capabilities should be supervised or assisted when using Grab Bars.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.
- The grab bar provides support and increases stability for an individual. The grab bar is not designed to support the total weight of an individual.

#### ⚠ INSTALLATION WARNINGS

- To ensure the user's safety, installation **MUST** be done by a qualified professional.
- After installation, **ANY** adjustments and **BEFORE** use, make sure all attaching hardware is tightened securely. **NEVER** use the Grab Bar if **ANY** looseness is detected.
- The Grab Bar **MUST** be installed so the mounting brackets are on wall studs. At least two of the three screws on each mounting bracket **MUST** attach through the wallboard or tile and secure into the wall studs. Special fasteners may be required if wall studs are not in the desired mounting location.
- The standard distance is 16-inches between centers of wall studs. Older homes do not always fall in line with current codes. Grab Bars can be installed vertically, horizontally or at an angle, but the screw holes of the grab bar **MUST** be aligned so at least two of the three mounting screws are secured into the wall studs
- When drilling into the wall, exercise care to avoid any electrical wiring or plumbing that may be located behind the wall. As older homes do not always fall in line with current housing codes and requirements, know where internal wall wiring and plumbing is located so there is no interference with installation.

#### ⚠ DANGER

- Damaged electrical wiring can cause electrical shock and fire.

### Installing the Grab Bar

NOTE: For this procedure, refer to FIGURE 1.

1. Use a stud finder to find the proper area for installation.
2. Put the grab bar against the wall and mark each mounting hole position with a pencil (3 holes per side, 6 total).
3. Put the grab bar aside.

NOTE: If installing the grab bar in a tiled area, a carbide drill bit will be required.

4. At each pencil marking, drill a 1/8-inch pilot hole into the wall and wall stud (3 holes per side, 6 total).
5. Put the grab bar against the wall and align the mounting holes with the drilled holes.
6. Thread a screw into each mounting hole (3 screws per side, 6 total). Hand tighten the screws.
7. Using a Phillips™ head screwdriver, securely tighten all mounting screws.

### Using the Grab Bar


Use the grab bar for support when entering or exiting the bathing area.

### Care and Maintenance

- Regular cleaning and inspection of the grab bar is recommended.
- Make sure that all attaching hardware is present and secure.
- To clean the grab bar, wipe the entire surface with a damp cloth and mild soap or non-abrasive cleaner.

\*NOTE: The wall stud width may vary, depending on the age of the home.

#### DETAIL "A" - REFERENCE DIMENSIONS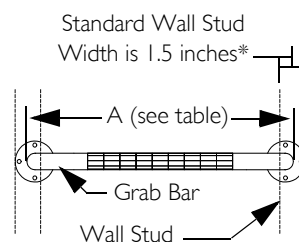

Grab Bar Model Number	A (Length)
312-3, 512-3	12"
316-3, 516-3	16"
318-3, 518-3	18"
324-3, 524-3	24"
332-3, 532-3	32"

#### DETAIL "B" - INSTALLATION CROSS-SECTION

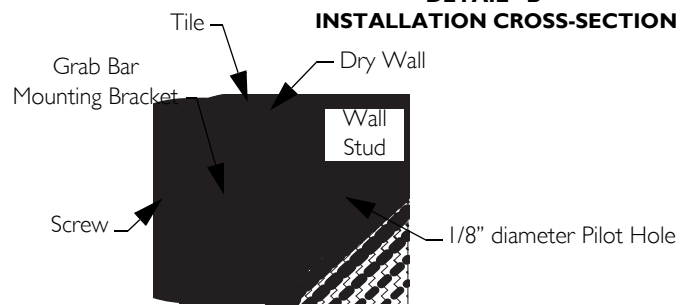
FIGURE 1 Installing the Grab Bar

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

---

# LIMITED WARRANTY

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN, THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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REMARQUE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

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© 2009 Invacare Corporation
Phillips is a registered trademark of the Phillips Screw Company